

SPECIALTY SIZES: 30% of Population

200S *Super Short*

1 Does your patient have a kyphotic (chin-on-chest) neck?



- Kyphotic.
- Chin-on-chest.
- Geriatric.
- Osteoporitic and / or ankylosing spondylitic.

200L *Stout*

2 Does your patient have a very large neck circumference?



- Very large neck circumference.
- Obese.
- "No-neck".
- Massive shoulders.

250 *XS*

3 Does your patient have a very short, thin neck?



- Very short thin neck.
- Principally female.
- More prevalent in the Asian population.
- Rare.

STANDARD SIZES: 70% of Population

500 *Tall*

4 Does your patient have a long, tall neck?



- Long, tall neck.
- "Swan neck".
- Young women.
- Adolescents.

400 *Medium*

5 Is your patient female?



- Female.
- Standard size for mature women.
- Thin, mature men.

300 *Small*

6 All other adults.



- Standard size for mature men.
- Ample or swollen women.

PEDIATRIC SIZES:

Patented design uniquely maintains airway and c-spine alignment of children during the growth transition from birth to adolescence. Always raise the child's thorax with the appropriate height padding to accommodate the occipital prominence.

7 Is your patient a child (12 years or under)? Pediatric sizes should be selected according to the Broselow - Luten scale, or by patient age.



Color Coding Kids
The Broselow - Luten System



P0 0 - 6 mos



P1 6 mos - 2 yrs




P2 2 - 6 yrs



P3 6 - 12 yrs

APPLICATION TIPS:

- Wrap sides of anterior half of collar **snugly** to neck and direct ends up towards ears.
- Velcro® straps *must* be symmetrically aligned - blue on blue. (see "A")
- Assess skin and check for redness while changing pads.
- Angulation Buttons (see "B") are adjustable to fine tune fit and relieve redness or pressure (turn arrow in direction you want to slide, slide to new position, turn arrow horizontal to lock).
- **Front:** *Up* to relieve chin tip pressure
Down to relieve mandibular pressure
- **Back:** *Up* to relieve occipital pressure
Up for predominately supine patient.
- Sternal Pad (see "C") may be removed for eating, excessive swelling, or for sleeping comfort.
- Retaining Strap should *not* be tight; may be removed by unbuttoning (save for patient use after discharge).

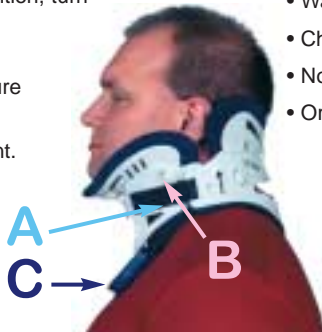

SKIN CARE TIPS:

- Change Sorbatex pads at least once per day, more if damp or soiled.
- Wash pads and plastic with mild soap and water.
- Rinse thoroughly.
- Roll pads in a towel to wring out excess water.
- Lay pads flat to air dry.
- Wash skin.
- Check for redness.
- No powder.
- Order Replacement Pads prn.

1MIPRC 4/02 © Jerome

MIAMI J® & MIAMI Jr.®

SIMPLE SIZING



A LATEX-FREE PRODUCT



QUESTIONS OR COMMENTS? CONTACT JEROME CUSTOMER SERVICE

305 Harper Drive, Moorestown, NJ USA 08057-3239

800-257-8440 • Fax: 856-778-5333 International: 856-234-8603 • Fax: 856-722-5487

Email: jeromemail@jeromemedical.com www.jeromemedical.com