

MIAMI J® & MIAMI Jr.® SIMPLE SIZING

A LATEX-FREE PRODUCT

SPECIALTY SIZES: 30% of Population

STANDARD SIZES: 70% of Population

PEDIATRIC SIZES:

200S
Super Short 10%

1 Does your patient have a kyphotic (chin-on-chest) neck?



- Kyphotic.
- Chin-on-chest.
- Geriatric.
- Osteoporitic and/or ankylosing spondylitic.

200L
Stout 15%

2 Does your patient have a very large neck circumference?



- Very large neck circumference.
- Obese.
- "No-neck".
- Massive shoulders.

250
XS 5%

3 Does your patient have a very short, thin neck?



- Very short thin neck.
- Principally female.
- More prevalent in the Asian population.
- Rare.

500
Large 10%

4 Does your patient have a long, tall neck?



- Long, tall neck.
- "Swan neck".
- Young women.
- Adolescents.

400
Medium 30%

5 Is your patient female?



- Female.
- Standard size for mature women.
- Thin, mature men.

300
Small 30%




6 All other adults.



- Standard size for mature men.
- Ample or swollen women.

Patented design uniquely maintains airway and c-spine alignment of children during the growth transition from birth to adolescence. Always raise the child's thorax with the appropriate height padding to accommodate the occipital prominence.


7 Is your patient a child (12 years or under)?*

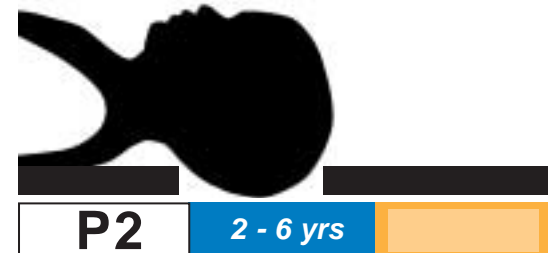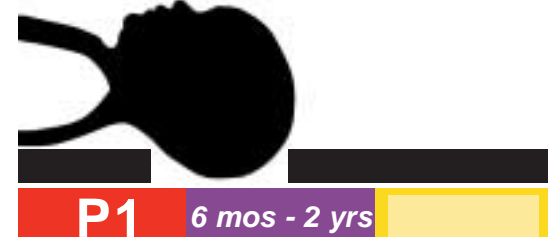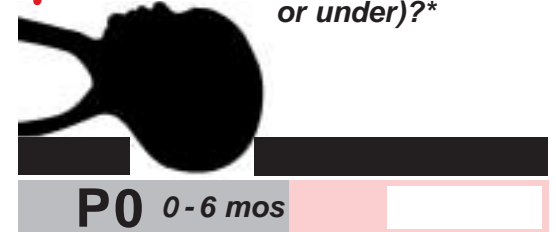

* Pediatric sizes should be selected according to the Broselow - Luten scale, or by patient age.

APPLICATION TIPS:

- Wrap sides of anterior half of collar **snugly** to neck and direct ends up towards ears.
- **Velcro®** straps *must* be symmetrically aligned - blue on blue. (see "A")
- Assess skin and check for redness each shift (while changing pads).
- **Angulation Buttons** (see "B") are adjustable to fine tune fit and relieve pressure (turn arrow in direction you want to slide, slide to new position, turn arrow horizontal to lock).
 - Front:** Up to relieve chin tip pressure
Down to relieve mandibular pressure
 - Back:** Up to relieve occipital pressure
Up for predominately supine patient.
- **Sternal Pad** (see "C") may be removed for eating, excessive swelling, or for sleeping comfort.
- **Retaining Strap** should *not* be tight; may be removed by unbuttoning (save for patient use after discharge).

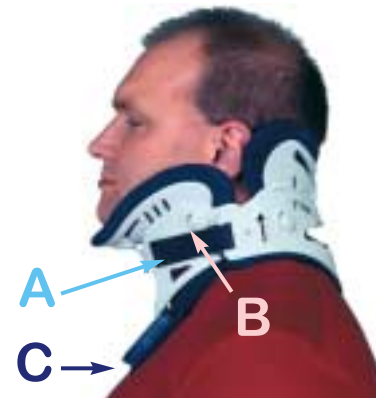

SKIN CARE TIPS:

- Change Sorbatex pads at least once per day, more often if damp or soiled.
 - Wash pads and plastic with mild soap and water.
 - Rinse thoroughly.
 - Roll pads in a towel to wring out excess water.
 - Lay pads flat to air dry.
- Wash skin.
- Check for redness.
- No powder.
- Order Replacement Pads prn.

www.jeromemedical.com

