









NECLOC® EXTRICATION COLLAR

SIZING & APPLICATION INSTRUCTIONS

1 Select proper size.


To be used only as directed by local Emergency Medical Service Authorities.

 Color Coding Kids
The Broselow - Lutens System

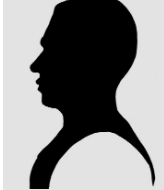
ADULT SIZES					PEDIATRIC SIZES		
NL-200EL <i>Stout</i>	NL-250E <i>XS</i>	NL-500E <i>Large</i>	NL-400E <i>Medium</i>	NL-300E <i>Small</i>		NL-P1 0 - 2 yrs	grey pink red purple yellow
NAVY	GREEN	LT. BLUE	ORANGE	YELLOW		NL-P2 2 - 6 yrs	white blue orange
						NL-P3 6 - 12 yrs	green

QUICK SIZE CHECK:

Use fingers to measure vertical chin-to shoulder distance.

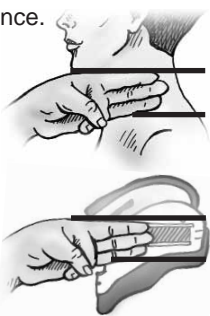


2 Assemble Front/Back Piece (where needed).

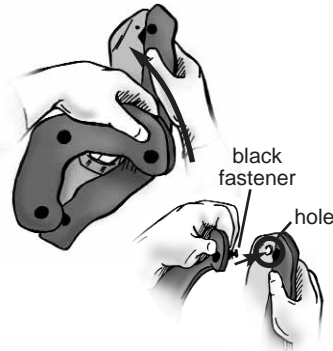

4 Apply Front. **SCOOP IT UP!**



Attach Retaining Strap.

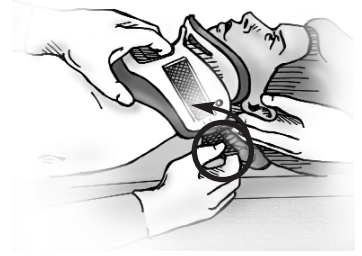

5 Apply Back symmetrically.




6 Tighten straps.



3 Use Cervical Spine Precautions.

INDIVIDUAL'S NEUTRAL ALIGNMENT:

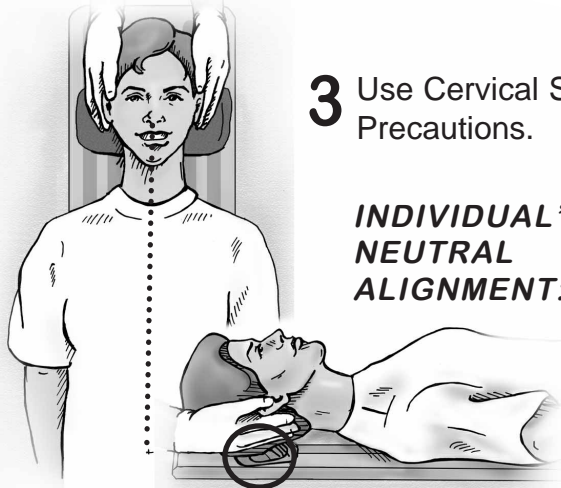

For children 12 years and under:

Maintain appropriate height padding under thorax to offset occipital prominence.



7 Check fit.



Too Small



Too Large



Just Right!

8 Cleaning.

Use a 1:10 dilution disinfectant : water



www.jeromemedical.com

305 Harper Drive Moorestown NJ USA 08057-3239 • 800-257-8440 • Fax: 856-778-5333



Authorized Representative: Scanlan Group B.V., Postbus 75664,
1118 ZS Schiphol Triport, The Netherlands.

JEROME
CERVICAL SPINE SYSTEM®

Helping You Meet The Challenge

1NLBAG 2/03 © Jerome